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Dear Customer,

We are pleased that you have decided in favour of this product. If you have any questions, please contact customer service via our e-mail address:

kundenservice@hsphanseshopping.de

Please read the instructions carefully before using the product for the first time and keep them in a safe place. If the product is passed on, these instructions must also be handed over. The manufacturer and importer accept no liability if the information in these instructions is not observed!

We reserve the right to change the product, packaging or accompanying documents at any time as part of our ongoing development programme.

MSA 30X is a sound amplifier and can be used as a hearing aid alternative. Please keep these instructions in a safe place for future reference. Due to continuous product improvement, the picture on the packaging may differ slightly from the actual product.

Please retain Instructions for future reference.

RECHARGEABLE MSA 30X

Rechargeable MSA 30X is a sound amplifier that helps amplify sound up to 30X, with no worries of having to replace batteries! It is equipped with a rapid charger and a rechargeable battery that can last up to 8 –10 hours of continuous use. It includes 6 interchangeable ear tips to SEAL OUT the noise!

Set includes:

- MSA30X - Rechargeable hearing amplifier
- MSA 30X quick charger
- MSA 30X cleaning brush
- 12 silicone earplugs
- USB charger (not shown)

Figure A

Cleaning Brush

Figure B


MSA 30X Charging Dock Station

Figure D

12 Silicone Adjustable Tips

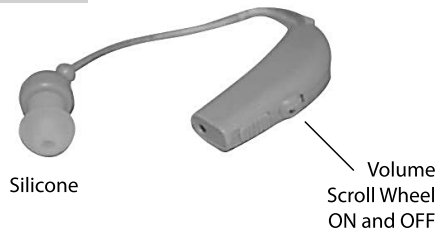Figure E

Silicone

Volume Scroll Wheel ON and OFF

CAUTION

- Rechargeable MSA 30X is NOT for use by children under 15.
- Keep out of reach of children and pets.
- The supply terminals are not to be short-circuited.
- Do NOT wet.

Note: Discontinue use of product in the event of irritation. It is recommended to consult a doctor before continued use of this product.

GETTING STARTED

It is important to follow the instructions (1-3) below for the first two times charging the Rechargeable MSA 30X Sound Amplifier:

1. Slide the switch down located on the back of the earpiece to turn the Rechargeable MSA 30X OFF.
2. Connect the power adapter to the charging station. Plug the power adapter into your normal power socket. You have done everything correctly so far, the green light on the charging station (see Figure B) lights up at the same time.
3. Place the Rechargeable MSA 30X into the recharging docking station. The green light on the docking station will turn red if connected properly.

Note: Do not remove the Rechargeable MSA 30X from the docking station until the light turns green (again) - signaling fully charged.

Important: Follow these steps to ensure longer life.

HOW TO WEAR THE RECHARGEABLE MSA 30X SOUND AMPLIFIER

Note: One (1) silicone ear tip is already on the Rechargeable MSA 30X Sound Amplifier.

- The Rechargeable MSA 30X includes 6 different silicone ear tips. Select the silicone tip that best fit the ear and place onto the Rechargeable MSA 30X Sound Amplifier.
- The Rechargeable MSA 30X can be worn on either the right or left ear. On the ear piece, make sure the volume tab is in position "1" before use.
- Below the volume scroll wheel is the ON/OFF switch. This must be in the "ON" position or in the "UP" position for use, end slide down for "OFF" to turn the device OFF .
- Take the ear piece and place it over the top of the desired ear. The ear piece will curve around end fit behind the ear.
- Carefully insert the silicone tip of the Rechargeable MSA 30X which is at the end of the small wire into the ear canal.
- Adjust the volume to the desired level.

Note: The device whistles if the volume level is too high. Adjust volume according to comfort level.

Note: Charging time will vary depending upon the level of battery power remainin prior to re-charge.

GENERAL CARE

- Clean Rechargeable MSA 30X daily with the provided brush to avoid unwanted earwax buildup.
- Brush the device back-and-forth across the ear bud (using the supplied brush) to remove wax and dirt deposited within the ear bud.
- Make sure the ear bud is clear of all debris and dirt.
- Wipe the Rechargeable MSA 30X with a clean, dry cloth only. Do NOT wet.

Note: There are no serviceable parts to this item.

STORAGE

When Rechargeable MSA 30X is not in use, clean it and store in a dry cool place.

TROUBLESHOOTING

Important information: It is recommended consulting a certified physician before deciding to wear Rechargeable MSA 30X end to follow up with constant checkups to ensure none of the listed conditions below apply.

The following are conditions under which one should NOT wear the Rechargeable MSA 30X Sound Amplifier. Do not wear the Rechargeable MSA 30X if:

- Have pain and discomfort in the ear.
- Have traumatic or visible congenital deformities on your ear.
- Have acute or chronic dizziness.
- Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days.
- Have unilateral hearing loss or sudden/recent onset within the last 90 days.
- Have significant build-up of wax or other materials in your ear canal.
- Bone conduction thresholds are better than the air conduction thresholds by 15dB or greater.

SPECIFICATION

Battery: 1,2V

Working Current: < 3mA

Signal to Noise Ratio: < 28db

Frequency: 100-4000Hz

Gain: 30db

Distortion: < 6%

DISPOSAL

In connection with the batteries and accumulators that we place on the market or with the delivery of devices that contain batteries or accumulators, we are obliged according to the Battery Act (BattG) to point out the following: The symbol of the crossed-out garbage can means that batteries and accumulators must not be disposed of with household waste. If the battery or accumulator contains more than 0.0005 percent by mass of mercury, more than 0.002 percent by mass of cadmium or more than 0.004 percent by mass of lead, the symbol below the crossed-out garbage can is expanded to include chemical symbols for the metals (Hg, Cd, Pb). As an end user, you are legally obliged not to dispose of all used batteries and accumulators with household waste, but to return them to retailers or municipal collection points. Old batteries and accumulators that are not enclosed by the old device, as well as lamps that can be removed from the old device, must generally be separated from the old device in a non-destructive manner before they are handed over to a collection point. Used batteries and accumulators can contain harmful substances that can harm the environment or your health if not stored or disposed of properly. Returned used batteries and accumulators are only sent for recycling. This means that valuable raw materials such as iron, zinc, manganese or nickel are recovered. You can either return used batteries or accumulators free of charge in your immediate area (e.g. in stores or recycling centers) or send them back to us free of charge. The environment and HSP Hanse Shopping GmbH thank you.

The crossed-out wheeled bin symbol means that this device is subject to the EU Waste Electrical and Electronic Equipment Directive (WEEE Directive). It indicates that the respective device must be disposed of separately from unsorted municipal waste at the end of its service life. A black bar under the crossed-out trash can indicates that the device was placed on the market after 2005. This device must not be classified as normal household waste, but must be collected separately and disposed of in an environmentally friendly manner at a municipal collection point. Owners of old devices from private households can hand them in free of charge at the collection points of the public waste disposal authorities or at the collection points set up by manufacturers or distributors in accordance with the Electrical Electronics Act. Shops with a sales area of at least 400 m² for electrical and electronic devices are subject to take-back. From July 1, 2022, those grocery stores with a total sales area of at least 800 m² that offer electrical and electronic equipment several times a year or permanently and make them available on the market will also be subject to this. This also applies to sales using means of distance communication if the storage and shipping areas for electrical and electronic equipment are at least 400 m² or the total storage and shipping areas are at least 800 m².

The recycling symbol consists of three arrows that represent the recycling cycle. The packaging and filling materials can be recycled. Do not dispose of the packaging and filling materials in household waste, but instead take them to the recycling collection in an environmentally friendly manner.

The CE marking confirms that the device complies with the applicable European requirements applicable to the respective device group and that the specific declarations of conformity are available.

WARRANTY

We assure that all products are made of high-quality materials and have been checked with the utmost care. However, if damage or a fault should occur during normal use, please contact the seller of the product. A prerequisite for the warranty is operation and proper assembly in accordance with the operating instructions.

Improper use and / or improper transport may invalidate the warranty.

The warranty period is 2 years and begins on the date of purchase. If the product you have purchased is not free from defects, please contact the seller of the product within the period from the date of purchase.

The following are excluded from the warranty:

- Damage due to external force
- Non-compliance with the operating instructions
- Repairs and modifications by persons not authorised by us
- Damage due to incorrect handling or maintenance
- Damage due to neglect or accidents
- Wear and consumables

IMPORTANT NOTICE

If you have any questions about the product or warranty claims, please contact our customer service. Do not send your product to our address without request. The cost and risk of loss for unsolicited mailings will be borne by the sender. We reserve the right to refuse acceptance of unsolicited deliveries or to return corresponding goods to the sender at the sender's expense

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