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Trapeze Bar RFM / Lifting Bar 100 kg max. load Standing model PZN: 3429465

Function

The Trapeze Bar is meant to assist in raising from the lying to the sitting position. Therefore the user must have enough remaining strength.

Indication:

- for raising in bed without aid.
- for raising onto a bedpan.
- for changing from bed to wheel- or toilet chairs.

Counter indication:

- People with known back problems must not use the Trapeze Bar.
- Apoplex patients and Hemiplegics must not use the Trapeze Bar.

Mounting

For mounting an even and firm surface of 86 x 76 cm is required.

Assemble the parts as shown in the graphics.

Before each use

It has to be checked whether the grip is safely placed between the locking pins.

Use

Place the Trapeze Bar freely at the head board of the bed. Carrier and grip have to be aligned with the bed.

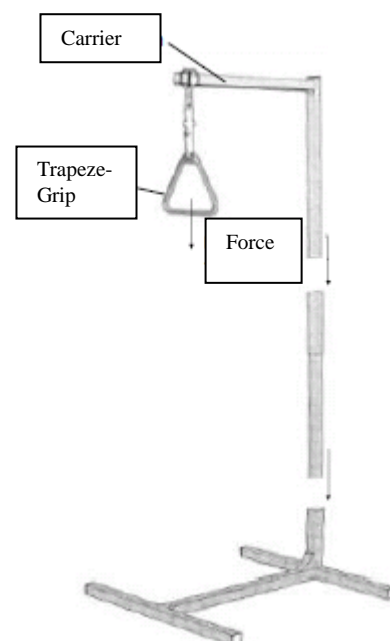
The grip has to be adjusted to the user. The force applied while raising has to go straight down.

Do not swing at the Trapeze Bar as to avoid injuries.

The grip should be above the user's chest. Do not use the Trapeze Bar for shifting higher onto the bed. For changing into a wheel- or toilet chair and aid should lend assistance. Do not let go of the grip while the aid is close by, as to avoid injuries to the aid's head.

Cleaning / Disinfection

Do not use sharp or abrasive tools or cleaners for cleaning. The painted parts are best cleaned with a wet cloth and mild cleaner. Normally cleaning the plastic parts with a wet piece of cloth is sufficient. Use special plastics cleaners, if you deem it necessary.

Disinfection

The Trapeze Bar can be disinfected using standard disinfectants, which are suited for plastics.

Note: Obey the manufacturer's instructions while using disinfectants.

Re-Use

Before each change of users the Trapeze Bar should be cleaned and checked for damages. Clean in accordance with the guidelines of the Robert-Koch-Institute (RKI) for hospital hygiene and prevention of infections.

Maintenance

Check monthly for any signs of wear and tear of the weldings and the trapeze grip. Please also check the grip band for signs of wear. Replace defective parts immediately. Replace defective parts immediately.

Safety

- The maximum load is 100 kg!
- Always align the Trapeze Bar with the bed.
- Do not swing at the Trapeze Bar!
- Do not use the Trapeze Bar for shifting higher in your bed.
- Only assemble as instructed!
- Use the Trapeze Bar only on firm and even ground!
- Do not use the Trapeze Bar, if any part is defective!
- Do not leave children alone in a room with a Trapeze Bar! It's no toy.
- Dismantle the Trapeze Bar to move or carry it!
- The Trapeze Bar can be used from 5 to 70°C!
- Do not attach any foreign parts to the Trapeze Bar!
- Do not drill into the metal frame or damage it otherwise!
- Keep this manual and let each user read it before the first use!

Technical Data

Area	82,5 x 75,5 cm
Total height	176 cm
Max. load	100 kg
Weight	11 kg
Color	grey

Contents: 1 Trapeze Bar with Trapeze Grip, 1 Manual

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Rehaforum MEDICAL GmbH