Actimo® Home

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DE

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Material Handgriff: Ethylvinylacetat (EVA)

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User Manual

EN

⚠ Please read the entire user manual carefully before assembling, adjusting, or using the rollator for the first time. Keep the user manual for later reference.

Purpose

The rollator is intended to support the walking of persons whose mobility is restricted due to illness or surgery. The rollator is intended for indoor use only.

The rollator has a tray as well as a net for transporting objects of daily use.

Indication

The rollator supports the preservation of activity and mobility and is suitable for

- slight restriction of mobility of the legs and knee joints
- slight loss of strength
- slight balance disorders.

Contraindication

The use of the rollator is unsuitable in case of:

- Perceptual disorders
- Severe balance disorders
- Reduced eyesight
- Loss of limbs in both arms
- Joint damage in the arms

Assembly

Your dealer will gladly assist you with the assembly.

1. Before unpacking, check the box for any damages. If the contents are damaged, contact your dealer immediately. The rollator is delivered fully assembled and tested by the dealer or support personnel.
2. Take out the rollator. Unfold the rollator by pushing down the upper black crossbar(see figure 1). Check at the lower crossbar if the folding mechanism is completely unfolded(see Figure 2).

Fig. 1

Fig. 2

3. Then insert the handle (see Figure 3) and adjust the height with knob to fixed it(see figure 4).

Fig. 3

Fig. 4

4. To adjust the handle height, loosen the two wing nuts on the frame by turning them counter-clockwise (see Figure 4). Remove the two screws one after the other and hold the handle. Adjust the height (see Figure 5) and insert the two screws from the inside through the frame holes. Tighten both screws clockwise.

Fig. 5

Fig. 6

5. Place the insert floor and gently press the tray onto the frame (see Figure 6).

6. To brake, pull the lower lever towards the handlebar (see Figure 7). To engage the parking brake, press the brake lever downward (see Figure 8). To release the parking brake, pull the brake lever upward (see Figure 9).

Fig. 7

Fig. 8

Fig. 9

7. To fold the rollator, remove the tray. Hold the rollator by the handle with the brake applied and pull the lower black crossbar upward (see Figure 10.).

Fig. 10

Safety Instructions

- Do not use a defective rollator!
- Do not use the rollator if the brakes are defective and test them for functionality before each use.
- Regularly inspect the rollator for wear and damage. Heavily worn tires can affect the rollator's braking function. Have damaged parts replaced by your dealer.
- Before each use, ensure that the rollator is fully unfolded.
- Check the tightness of the screws and correct height adjustment before each use.
- The rollator is not a wheelchair or transport vehicle but solely a walking aid.
- The Actimo® Home is only intended for indoor use.
- Use the rollator only on firm, level ground.
- Do not carry the rollator by removable parts or brake cables.
- Exercise caution when folding and unfolding the rollator - there is a risk of pinching!
- Before each use, ensure that the rollator is fully unfolded.
- The tray and the underlying net can each be loaded with a maximum of 5 kg.
- Do not hang bags on the handles. There is a risk of tipping!
- The maximum user weight is 100 kg.
- Do not make structural changes to the rollator without prior consultation with your dealer or Rehaforum Medical GmbH.

Cleaning

You should clean or disinfect the rollator at regular intervals (mandatory when changing users). Use a mild soapy solution without abrasive additives and a soft cloth. You can disinfect it with a commercial product according to the manufacturer's instructions. Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!

Ensure that no dirt accumulates on the tires or brakes. This can be removed with a soft brush and warm water.

Maintenance

The rollator should be serviced annually by your dealer.

This involves checking the stability of the frame, the strength of the mountable parts, and ensuring that all screws are securely fastened and, if necessary, adjusted.

All screws must be tightened. Worn, damaged, or broken parts must be replaced immediately.

Malfunctions:

If you notice a decrease in braking power or difficulty in wheel movement, contact your dealer.

If you notice damage or rust on the frame, notify your dealer immediately and do not use the damaged rollator!

Re-Use

The rollator is suitable for re-use. Before transferring it to another user, the rollator must undergo an inspection, cleaning, disinfection, and maintenance by a specialist retailer.

If parts of the rollator are damaged or destroyed, they must be replaced with new parts. It is recommended to replace the wheels with each change of user. Tighten all loose screws and nuts.

The product must be in perfect condition before re-use.

It is prohibited to re-use the rollator if irreparable damage is found, such as a bent frame.

The user manual must be provided with the rollator for each user change.

Scope of delivery

Actimo® Home

Tray

Net

User Manual

Technical Specifications

Weight: 6.2 kg;

Frame Material: Aluminum (corrosion protection - anodic oxidation)

Handle Material: Ethylene Vinyl Acetate (EVA)

Tire Material: PVC 6"

Maximum Load: 100 kg

Width: 58 cm

Depth: 60 cm

Adjustable Height: approx. 76 - 86 cm

Dimensions when folded: approx. 52 x 51 x 21 cm

Outer Dimensions (Packaging): approx. 54 x 53 x 23 cm

Storage

Store your rollator in a place protected from light, dust, and moisture, and protect it from strong heat sources.

Disposal

Do not dispose of the rollator as regular household waste. Please contact your dealer or take the rollator to your local disposal site.

Service Life

If the rollator is used within its intended purpose and maintained, inspected, and serviced according to the instructions, a lifespan of five years can be expected for the Actimo® Home. However, the lifespan can vary significantly depending on the treatment and care of the product. A particularly careful handling of the product can extend the lifespan, while extreme and improper handling can shorten it.

The determination of the lifespan by Rehaforum Medical GmbH does not constitute an additional warranty.

Warranty

The warranty period is 24 months and applies to all product defects that can be proven to be due to manufacturing or material defects.

In particular, the warranty does not apply if the rollator has features which indicate that it has been repaired or otherwise interfered with by unauthorised dealers, or if third-party accessories have been installed in the rollator without the prior written consent of Rehaforum Medical GmbH.

Reporting/Incidents

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the member state where the user and/or patient is located.

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Fastlæggelsen af levetiden af Rehaforum Medical GmbH udgør ikke en yderligere garanti.
Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn